

How to Use the EpiPen Auto-Injector

Blue to the sky. Orange to the thigh.

How to use EpiPen® and EpiPen® Jr (epinephrine) Auto-Injectors. Remove the Epi Pen Auto-Injector from the carrier tube and follow these 2 simple steps:



- **Hold firmly with orange tip pointing downward.**
- **Remove blue safety cap by pulling straight up. Do not bend or twist.**



- **Swing and push orange tip firmly into mid-outer thigh until you hear a /click**
- **Hold on thigh for several seconds.**



Built-in needle protection

- After injection, the orange cover automatically extends to ensure the needle is never exposed.